



OrthoHeal

Healing comfortably



www.orthoheal.com



About OrthoHeal

www.orthoheal.com



OrthoHeal is the leading Med-Tech company that specializes in the innovative orthopedic technology & devices with a global footprint in over 50 countries



OrthoHeal
Healing Comfortably

FlexiOH® is the flagship brand of innovative light cured polymer immobilization technology developed in 2015 & commercially launched in 2019, that introduced a new league of orthopedic immobilization alternative to traditional casts

It is indeed, one-of-a-kind orthopedic immobilizer that blends precision, rigidity, washability and ventilation without compromising comfort, safety, hygiene and convenience of post-fracture healing experience

FlexiOH® is developed, manufactured, patented and delivered by OrthoHeal across the world



About FlexiOH

www.orthoheal.com



FlexiOH[®]
Light Cured Polymer Immobilizer



FlexiOH[®] is the first of its kind that has the rigidity to hold the fractured part as well as ensuring proper skin ventilation

Comes as an easy to fix wearable material on the injured part

A light treatment completes the curing process and makes the FlexiOH[®] rigid

Process is quick and only takes 3-10 minutes, depends on the size

When zipper clasp removed, patient will be unable to unzip and remove cast



About RizyCure

www.orthoheal.com



RizyCure[®]
Curing Light Device - Corded



FlexiOH[®] comes as soft flexible which is stretch to fit.

RizyCure[®] corded light device which is specially designed to cure FlexiOH[®].

It is high intensity blue light (not a UV light).

It takes only 3-10 minutes to cure a FlexiOH[®] variant



FlexiOH Features

www.orthoheal.com



FlexiOH[®]

Light Cured Polymer Immobilizer

Using unique combination of design and advanced material science

KEY FEATURES



Waterproof



Breathable



Comfortable



How FlexiOH Works?



Choose size and wrap it

1



After choosing the size, place the FlexiOH® over the fractured hand anchoring thumb

Just Zip it up

2



Lock the zip and pull the slider of the zip across the FlexiOH®

Cure it with RizyCure® light and it's done.

3



Expose the FlexiOH® with RizyCure® for 3-4 mins thoroughly till every corner hardened.


Cut the zipper clasp to lock it.

4



Cut the zipper clasp to lock the FlexiOH® so that patient can't open.


Competition Landscape

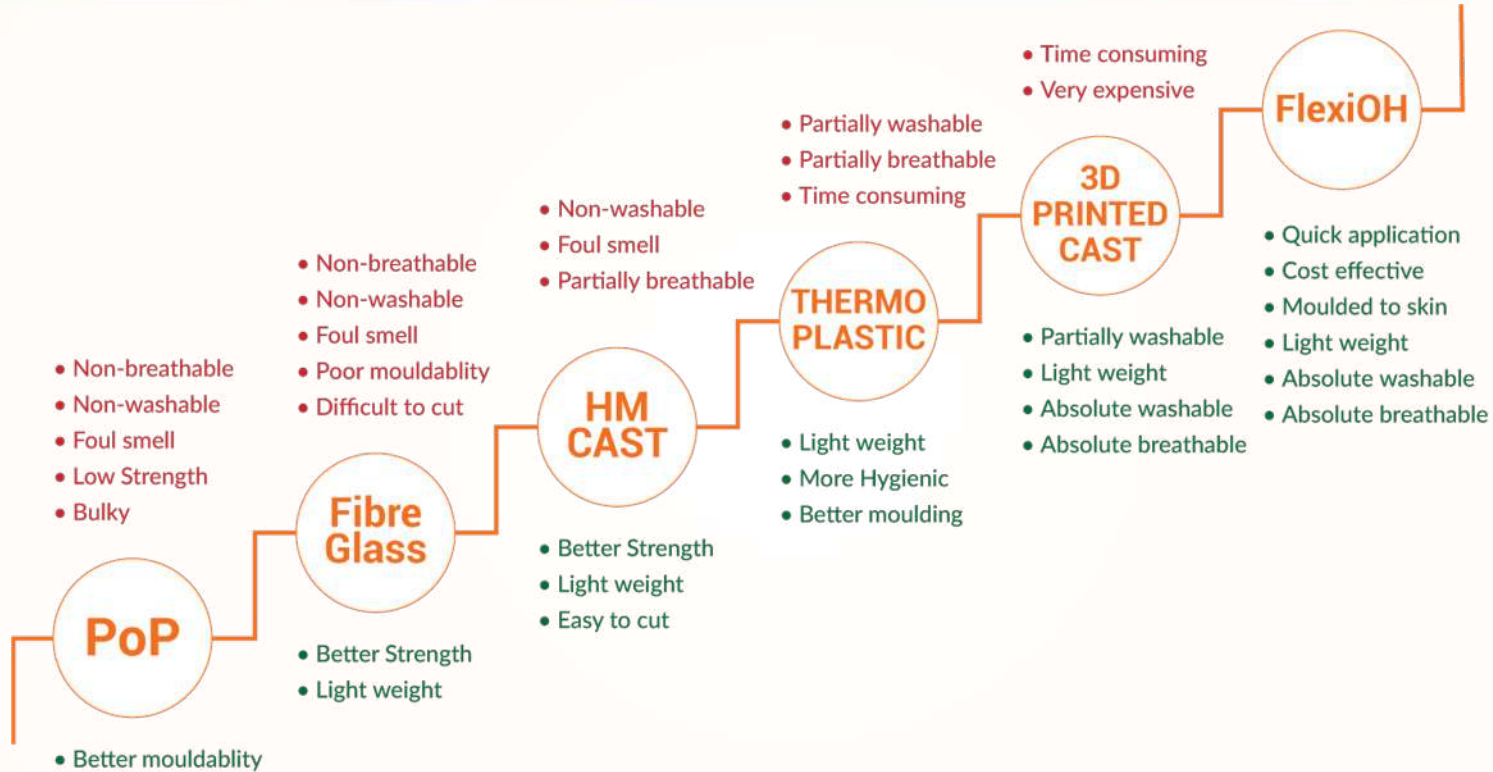


Why FlexiOH?




Traditional Braces

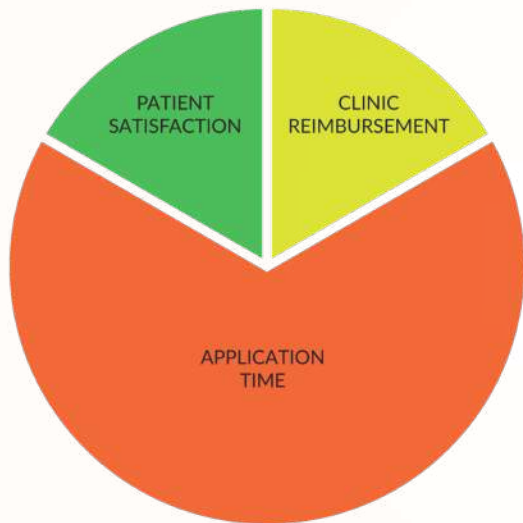

FlexiOH®

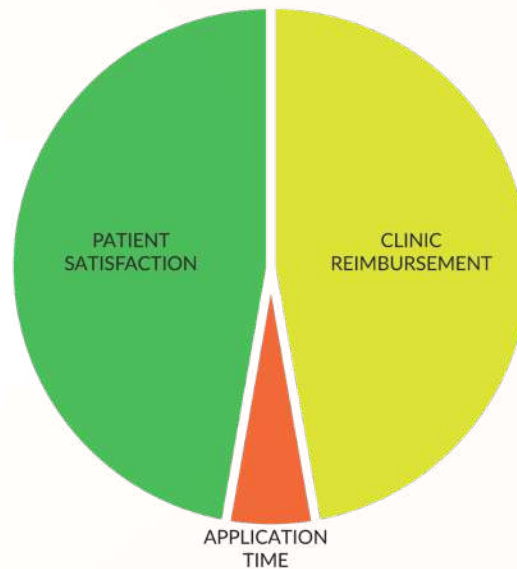

Application time & Reimbursement

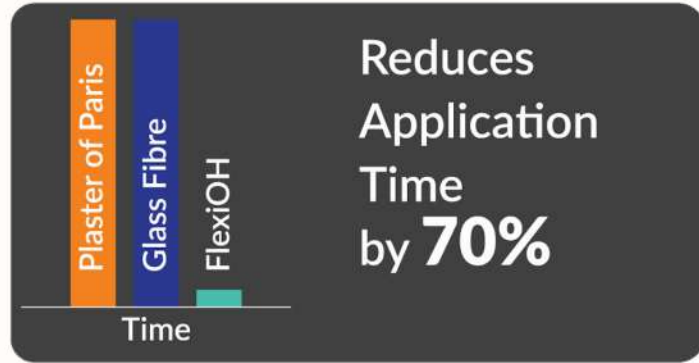

NOTE: All pictures shown are for illustration purpose only. It is not intending to compare any specific brand / design / company.



FlexiOH Variants


www.orthoheal.com



Covers 80% of all possible fractures in pediatric sizes.

Covers 75% of all possible fractures in Adults.

Comes in 8 Variants with different 39 sizes

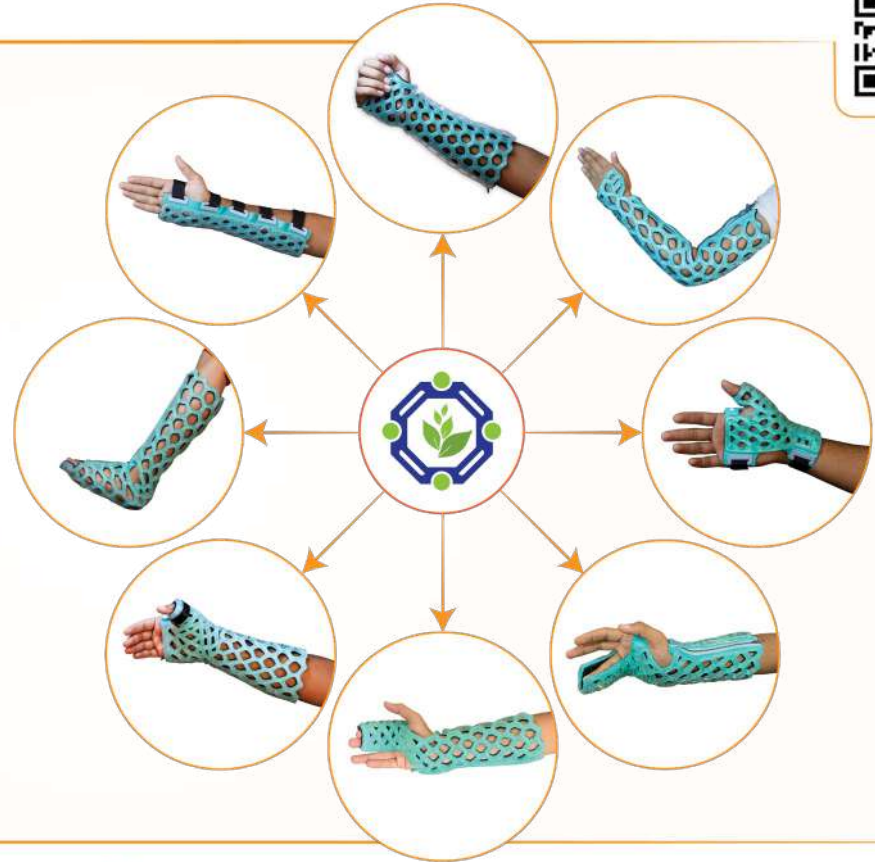

Product Offerings

www.orthoheal.com



FlexiOH® Short Arm

Size Available : XS, S, SM, M, SL, L, XL

HCPCS Code : L3982, L3984

ASP Price : \$ 115



FlexiOH® Full Arm

Size Available : 2XS, XS, S, M, L

HCPCS Code : L3980, L3981, L3762

ASP Price : \$ 225



FlexiOH® Thumb Spica

Size Available : S, M, L

HCPCS Code : L3923, L3924, L3917, L3984

ASP Price : \$ 50



FlexiOH® Long Thumb Spica

Size Available : S, M, L

HCPCS Code : L3984, L3917, L3918

ASP Price : \$ 115

Product Offerings

www.orthoheal.com



FlexiOH® Radial Gutter

Size Available : S, M, L

HCPCS Code : L3982

ASP Price : \$ 115



FlexiOH® Ulnar Gutter

Size Available : S, M, L

HCPCS Code : L3982

ASP Price : \$ 115



FlexiOH® Short Leg

Size Available : 2XS, XS, S, M, L, XL

HCPCS Code : L2116

ASP Price : \$ 225



FlexiOH® Splint / Slab

Size Available (in Inches) :
3x9, 3x12, 3x15, 4x12, 4x15,
4x18, 5x12, 5x15, 5x18

HCPCS Code : L3982

ASP Price : \$ 115

Product Offerings

www.orthoheal.com



RzyCure®
Curing Light Device - Corded

Size Available : NA

HCPCS Code : NA

ASP Price : \$150



FlexiOH®
Double Action Pliers

Size Available : NA

HCPCS Code : NA

ASP Price : \$50

Our Presence



50+

Countries

1000+

Prescribers



The Recipient

www.orthoheal.com



Thank You



EuroAmerican

Specialized Medical Products

Distribution By:

EuroAmerican IP, LLC
2511 NW 16th Lane, Bay #2
Pompano Beach, FL 33064
USA

Sales Director: Tami Dodge

Email: Tami@EuroAmericanProducts.com
Call: +1 (954) 257 9591
Web: www.EuroAmericanProducts.com

